

# HOCL

solutions

*The New Generation of Clean!*

WOUND **+** SKIN  
SOLUTION  
GEL

[www.HOCLsolutions.com](http://www.HOCLsolutions.com)

NET WT. 2 FL. OZ. [59 ml]

## Doctor & Pharmacist Recommended

Safe for the management of minor wounds  
and minor irritations caused by:

- Dry Skin
- Sunburns
- Cuts
- Insect Bites & Stings
- Burns
- Plant Saps & Oils
- Blisters
- Scratches
- Rashes
- Scrapes



**ALCOHOL  
FREE**



**PEEL HERE**

V2

**INGREDIENTS:**

Electrolyzed Oxygenated Water (H<sub>2</sub>O) - 97.742%  
Sodium Magnesium Fluorosilicate - 2.2%  
Sodium Chloride (NaCl) - 0.048%  
Hypochlorous Acid (HOCl) - 0.01%

**INDICATIONS:** Wound + Skin Solution is for immediate care of minor cuts, minor scrapes and minor burns.

**DIRECTIONS FOR USE:** Apply Wound + Skin Solution directly on the wound to flush away dirt and debris. Thoroughly clean the wound. If dressing is used, saturate with Wound + Skin Solution and apply directly over the wound. Dressing should extend beyond the wound margins. Flush the wound with solution and apply a fresh moistened dressing one or more times daily as needed. Soak adherent dressing with solution to ease removal.

**DISTRIBUTED BY:**

HOCL Solutions, LLC  
2172 N. Fox Hollow Dr. Suite 1  
Nixa, MO 65714

[www.HOCLsolutions.com](http://www.HOCLsolutions.com) | [info@HOCLsolutions.com](mailto:info@HOCLsolutions.com)

**WARNING:** Not for injection.

In case of deep or puncture wounds, animal bites or serious burn, consult a health care professional. If condition persists or worsens for long than 7 days consult a health care professional.

**KEEP OUT OF REACH OF CHILDREN.**

**STABILITY:** Product has a shelf life up to 18 months.

**STORAGE:** Do not allow product to freeze.

**DISPOSAL:** No special disposal considerations required.

